

Faeces Collection Kit for Enteric Pathogens

www.sapathology.sa.gov.au


This test may include bacterial culture, viruses, *C.difficile* toxin, *Helicobacter pylori* antigen detection and parasites.



Do Not contaminate specimen with urine.

How to collect

1. On the label of the brown top container write your:
- full name, date of birth, date and time of collection.
2. Place a clean icecream container or similar in the toilet bowl and pass a bowel motion.
3. Use the spoon in the lid of the brown top container to transfer one full scoop of faeces from the ice cream container and tighten the lid.

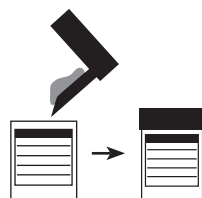

Delivering the specimen

Place the container into the zip-loc section of the plastic transport bag.

Put the SA Pathology Request Form in the open pocket of the bag.

Refrigerate the specimen until delivery. Deliver the specimen to your nearest SA Pathology Patient Centre (*see back of Request Form*).

Your results

Your doctor will advise you when results are available.